
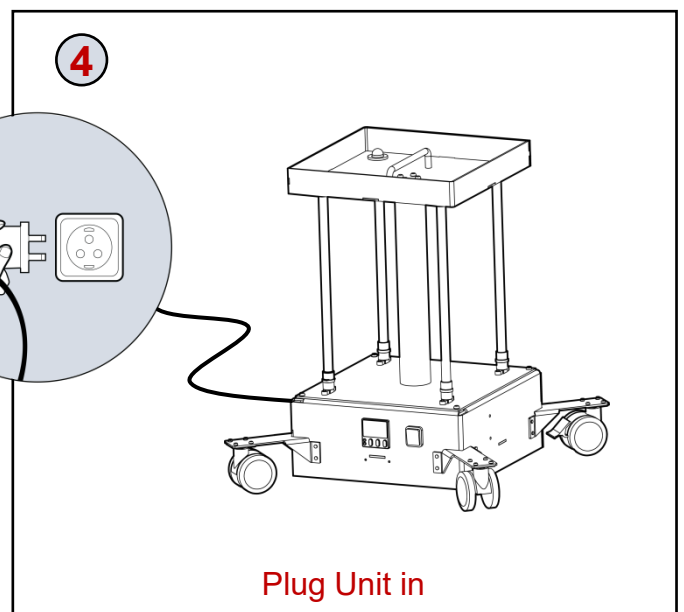
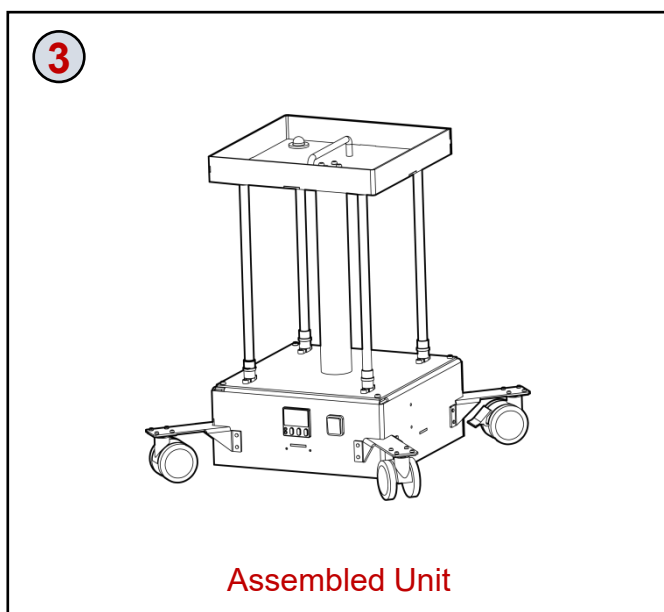
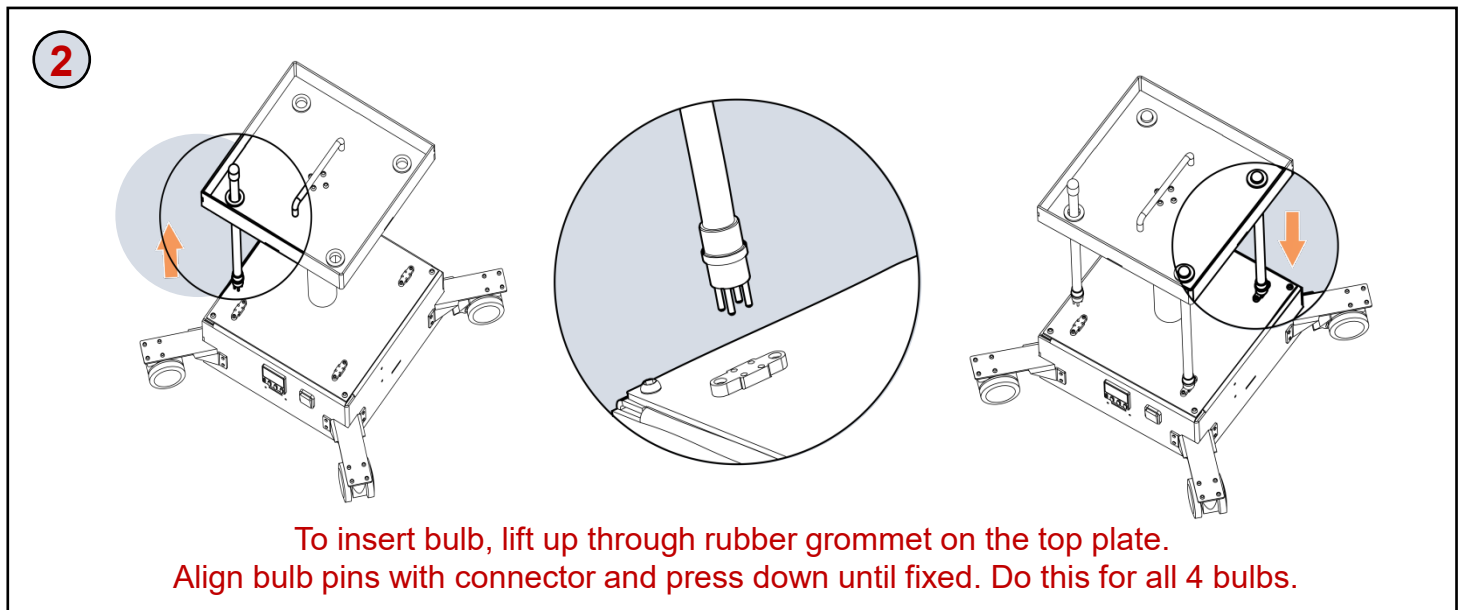
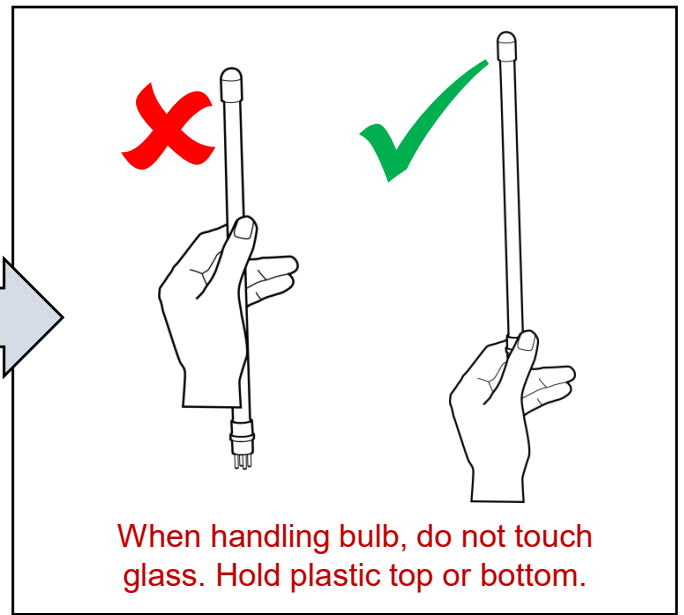


# MUVS OPERATING INSTRUCTIONS

The Mobile, Portable  
UV-C Room  
Sterilizer

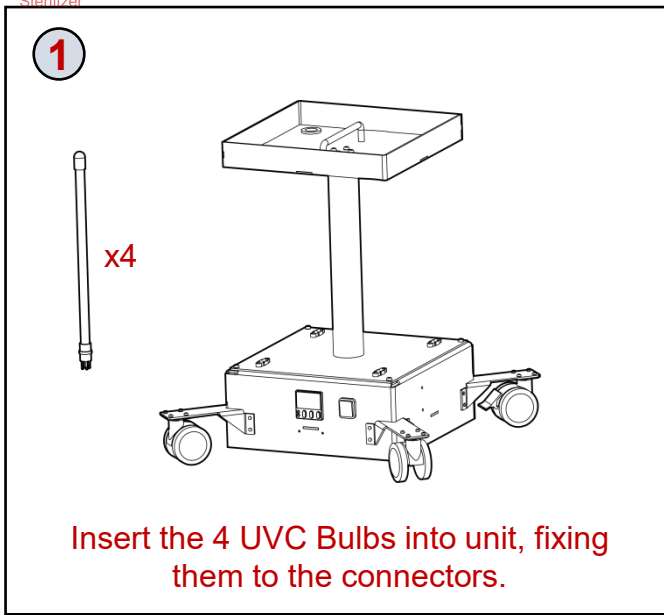


# MUVS OPERATING INSTRUCTIONS

The Mobile, Portable  
UV-C Room  
Sterilizer




5

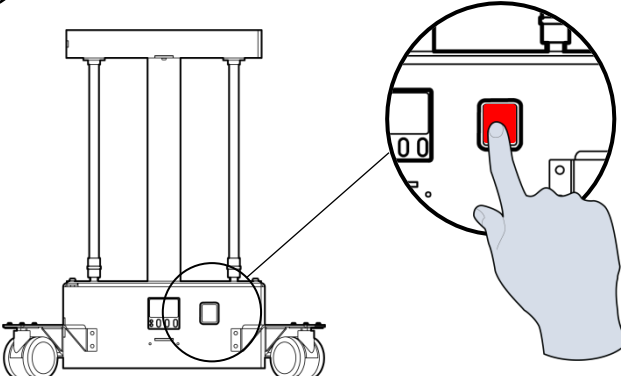

Power the unit on.  
(The LCD on the timer will light up, indicating the unit is on)


6

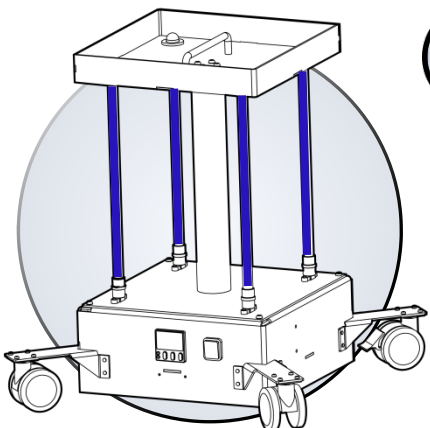

2 mins


Once the unit is powered on, you have 2 minutes to set the timer before sterilization begins.

7

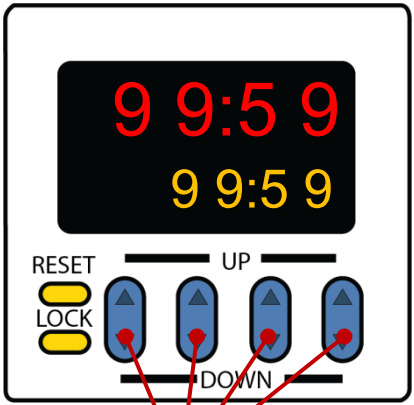


Adjust time using UP/DOWN buttons  
(Max Run Time 99mins 59secs)

8

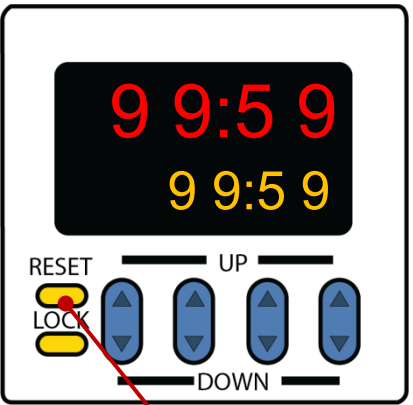


Once timer is set press the RESET button  
(Failing to press the RESET button causes the unit to default to original setting)


9

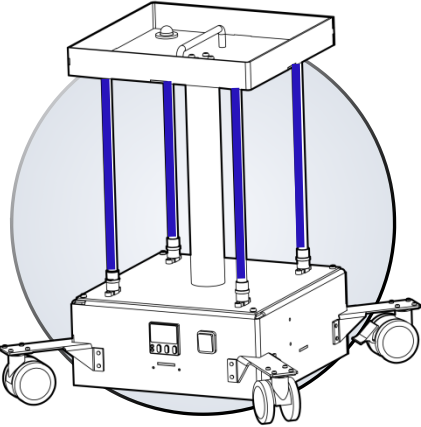

The MUVS bulbs will light up 2 minutes after switching the power button on. The cycle will last for the amount of time set on timer.

10

**WARNING** 

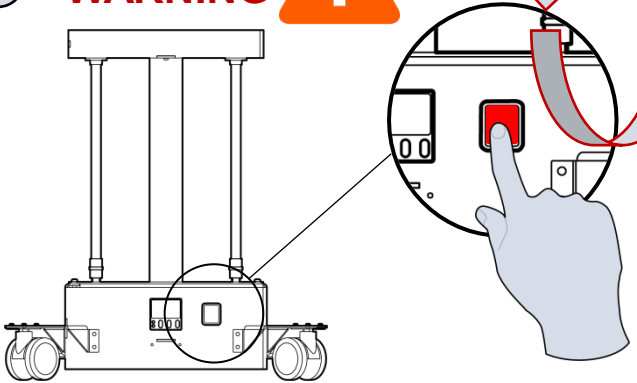

The MUVS must be powered off after each sterilization cycle to permit reset of the 2 minute countdown timer for next sterilization cycle.

AVT-201001.03